



Shanghai Gibo Star International Trade Co., Ltd

Add: Room 1301-1302, Ji Mo Road, Pu Dong New Area, Shanghai, PR. China, 200120

Tel: 86 21 5882 9477

Fax: 86 21 5887 5235

www.heatsupport.com

Battery Unit Usage Instruction

A. Li-polymer battery

1. Press ON/SETUP/OFF for two seconds until the LED turns red, the output power is 100% and the temperature reaches to 45°C-50°C.



2. Press ON/SETUP/OFF again when the LED turns orange, the output power is around 75% and the temperature reaches 40°C-45°C.



3. Press ON/SETUP/OFF for the third time when the LED turns green, the output power is around 50% and the temperature reaches 35°C-40°C.



4. Press ON/SETUP/OFF for the fourth time when the LED flashing, it is in the model of power saving.
5. Press ON/SETUP/OFF for the fifth time when the LED turns off, the system will be power off automatically.

B. Charge instruction:

When the battery is charging, LED on both battery and charger turns on red. And when LED on charger shows green, please keeping charging the battery until LED on battery turns GREEN.



Shanghai Gibo Star International Trade Co., Ltd

Add: Room 1301-1302, Ji Mo Road, Pu Dong New Area, Shanghai, PR. China, 200120

Tel: 86 21 5882 9477

Fax: 86 21 5887 5235

www.heatsupport.com

Important Notice:

1. Not allow to put the Li-polymer battery into water, fire or in a high-temperature environment. Not allow to punch or scratch lithium-ion battery. Not allow to connect the two ends of the battery to cause a short circuit. Don't take apart, re-assemble, repair battery, heater and charger to avoid breaking of the components.
2. Don't leave it on high temperature for long time to avoid low-temperature skin burning.
3. No permitted to use by the pregnant, baby and the non-vocal disabled. Not permitted to use by serious heart disease patient and carriers of artificial heart pacemaker. Not permitted to use by the blood-bleeding person or the person of bleed symptoms.
4. Don't pierce heater with sharp object.

Production maintenance & storage:

1. For the first use, please charge the battery to full level. After the LED light turns on green, don't charge for long time any more.
2. The battery will be run out if not using. **Special suggestion:** please charge the battery every 3-5 months and place in cool environment.
3. Battery set; heater and charger should be place away from water. Please take off battery, heater before washing the product and make sure to connect the power to the product until it is dry.
4. When pull the connector out of socket, please hold the connector tightly. Don't stretch the wire to break or destroy the plug.
5. Please place the product in a drafty and dry environment. The relative humidity is less than 80%, no corrosive gas.